
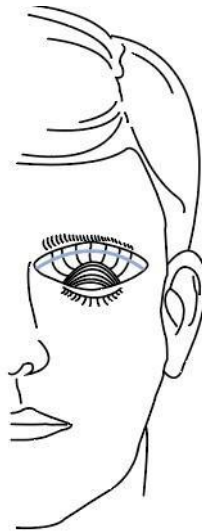


NICTAVI

Now we offer the Tarsus Eyelid Patch for eyelid closure.

EASY TO USE

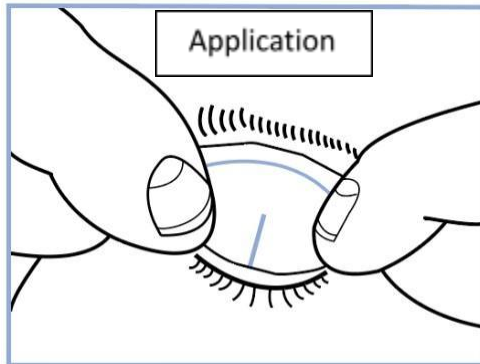

Eyelid surface must be clean and dry before applying patch.

USES: For day or nighttime use, for up to 12 hours, to sustain eyelid closure.

Recommended Single Use

When the Tarsus Patch is applied to the closed upper eyelid, the adhesion conforms the patch to the curvatures of the eyeball, on the eyelid, and the under brow, it acquires form and structure, holding the eyelid in the closed position in resistance to the retraction of the levator muscle.

FACTS: Eyelid closure retains moisture on the eye, reduces the chance of infection, protects and aids in healing the cornea. Tears deliver nutrients to the cornea and keep the surface of our eyes healthy with natural antibiotics called "lysozymes".

1. Remove the non-adhesive sticker along the division between the arrows.
2. Place centered on the closed upper eyelid, above the eye opening and lashes. Gently rub to adhere.
3. **WARNING:** Skin damage may occur when medical adhesives are not removed gently. To remove, gently run your finger along either side edge of the patch, rolling a portion backward. Grip the portion with your fingers and gently remove the patch. Painless removal with no annoying residue.

MATERIALS: Polyolefin plastic, polyester, sticky acrylate designed for medical/surgical use, hypoallergenic soft medical-grade silicone adhesive. **DO NOT USE IF ALLERGIC.**

Ask for the Tarsus Eyelid Patch for eyelid closure by Nictavi.

Cleanroom manufactured by Hi-Tech Products for Nictavi LLC in the USA. "Non-Sterile" Registered with FDA:

#3017733452, US Patent: #10,076,448, US Patent: #10,799,395, NPI: #1104411701

NICTAVI 1800 Vine Street, Los Angeles, California, 90028 USA.

